QUICK GUIDE, GETTING STARTED

PARI has designed this Quick Guide as a ready reference to keep you on the go.

- 1 Read the "Instructions For Use" manual thoroughly before operating unit.
- 2 If using the battery, make sure the battery is fully charged by checking the battery capacity indicator located on the front of the DURA-NEB 3000.

BRIGHT GREEN	Battery fully charged.
YELLOW	Approx. 1/3 battery capacity remaining.
RED	Recharging is a must (Only 1-2 treatment are left).

If using a wall outlet or 12V vehicle outlet, make sure all cords are connected properly

- 3 Pour medication into the nebulizer cup according to your prescription.
- 4 Assemble the nebulizer as described in the Instructions For Use.
- 5 Attach optional mask as instructed by your physician.
- 6 Attach the nebulizer tubing to the air outlet connector on the front of the DURA-NEB 3000.
- 7 Turn on the compressor.
- 8 Breathe in the medication mist through the mouthpiece or mask as instructed by your physician.
- 9 After your treatment is completed, turn compressor off and disconnect the nebulizer tubing from the air outlet connector. Clean and/or disinfect the nebulizer cup as shown in the Instructions For Use.

TEAR OFF AND FOLD QUICK GUIDE, THEN PLACE INTO CARRYING CASE FOR REFERENCE.

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