VORTEX™

Non Electrostatic Valved Holding Chamber

Specialists in effective inhalation

SINGLE PATIENT USE
LATEX FREE ■ DISHWASHER SAFE
SAVE THESE INSTRUCTIONS

CAUTION

U.S. Federal Law restricts this device to sale by or on the order of a physician. Use only when prescribed by a physician and with the prescribed medication. Read all warnings and instructions before use. Use the product only for its intended use as described in these Instructions For Use. Improper use can result in serious or fatal illness/injury.

WARNING

1. Close supervision is necessary when this product is used by children or near the physically or mentally impaired. This device contains small parts that may present a choking hazard to small children. Always keep out of the reach of children three years old and under. All parts should be seated firmly in place. Use of an improperly assembled PARI VORTEX™ Non Electrostatic Valved (N.E.V.) Holding Chamber could prevent adequate delivery of medication. If not sure of the suitability and/or functioning of your MDI, consult MDI instructions and/or your physician.
2. To reduce the risk of infection, illness or injury from contamination and to reduce the risk of improper performance and treatment, you must thoroughly clean and dry all parts of the PARI VORTEX™ as instructed on reverse side.
3. To reduce the risk of improper treatment, use only PARI accessories (e.g. masks).
4. To reduce the risk of improper treatment, product should be replaced if valves or parts are damaged or missing.
5. To reduce the risk of improper treatment, make sure duckbill valve is properly fitted.
6. If unsure on how to use this product, talk to your physician or pharmacist.
7. Administer medication per MDI instructions.

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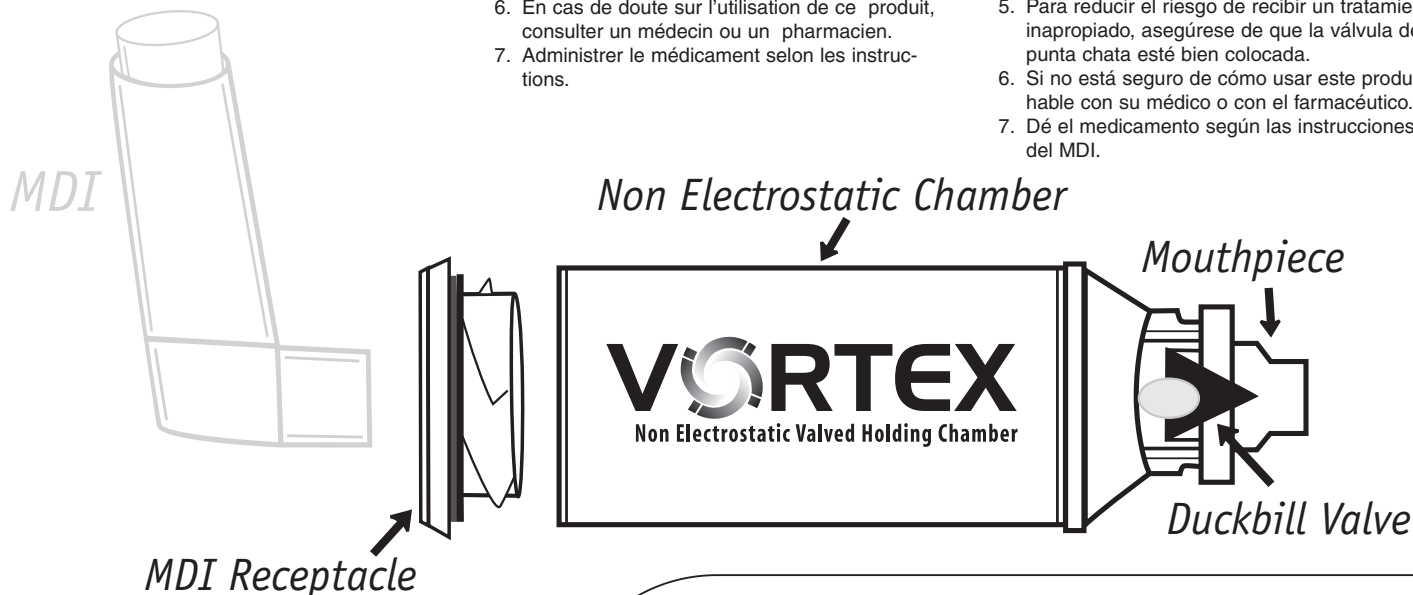
PARI VORTEX™ N.E.V. HOLDING CHAMBER has a 6 month limited warranty for home use.

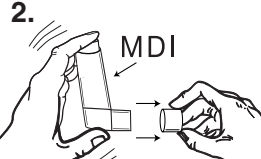
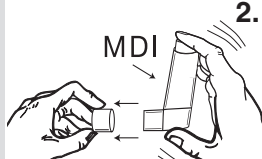
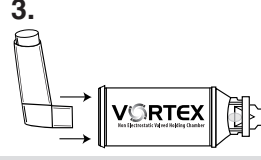
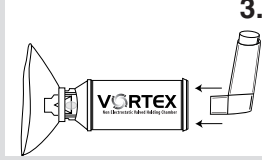
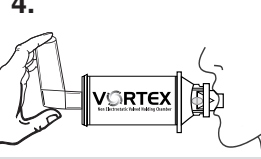
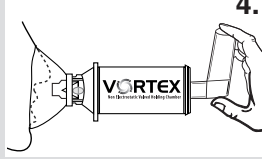
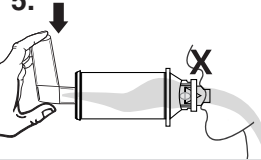
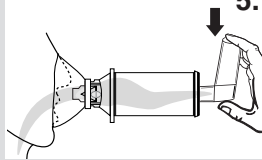
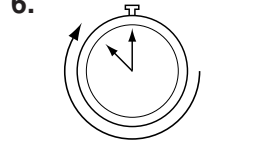
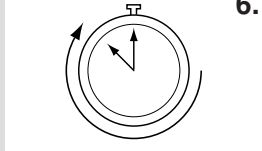
PARI RESPIRATORY EQUIPMENT, INC.
3 Justin Court, Monterey, California 93940
Tel 831.658.0901 / Fax 1.800.727.4112
(USA and CANADA) 1.800.FAST.NEB (327.8632)

www.PARI.com
E-mail ProductInfo@PARI.com
Respiratory CEU programs available
© 2002 PARI Respiratory Equipment, Inc.
Patents Pending Part #51D5000 Rev A 04/02

INSTRUCTIONS FOR USE - PARI VORTEX™ Non Electrostatic Valved (N.E.V.) Holding Chamber

IMPORTANT: Follow instructions included with your metered dose inhaler (MDI)

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